REPROCESSING OF REUSABLE MEDICAL DEVICES

- FR RETRAITEMENT DES INSTRUMENTS MÉDICAUX RÉUTILISABLES
- DE AUFBEREITUNG WIEDERVERWENDBARER MEDIZINISCHER GERÄTE
- IT RITRATTAMENTO DI STRUMENTI MEDICI RIUTILIZZABILI
- ES REPROCESAMIENTO DE APARATOS MÉDICOS REUTILIZABLES
- CS OBNOVA ZNOVUPOUŽITELNÝCH ZDRAVOTNICKÝCH POMŮCEK
- DA GENBEARBEJDNING AF GENBRUGELIGE MEDICINSKE ANORDNINGER.
- NL RECYCLAGE VAN HERBRUIKBARE MEDISCHE APPARATEN
- ET KORDUVKASUTUSEGA MEDITSIIINILISTE INSTRUMENTIDE TAASTÖÖTLEMINE
- F UDEELLEENKÄYTETTÄVIEN KLINISTEN LAITTEIDEN UDEELLEENKÄSITTELY
- EL ΕΓΑΝΕΤΕΕΡΡΑΣΙΑ ΕΓΓΑΝΑΦΗΜΟΤΗΙΜΩΝ ΙΑΤΡΙΚΩΝ ΣΥΣΚΕΥΩΝ
- HU ÚJRAFEELHASZNÁLHATÓ ORVOSI ESZKÖZÖK FELDOLGOZÁSA
- LV VAIRĀKKĀRT LIETOJĀMO MEDICĪNISKO IERĪCŪ PĀRSTRĀDE
- LT PAKARTOTINAI NAUDOJAMOS MEDICINOS ĮRANGOS PERDIRBIMAS
- MT RIPROĊESSAR TA' GHODDA MEDIKA LI TISTA' TERĠA' TINTUŻA
- NO BEHANDLING AV GJENBRUKBARE MEDISINSKE INSTRUMENTER
- PL OBRÓBKA INSTRUMENTÓW MEDYCZNYCH WIELOKROTNEGO UŻYTKU
- PT REPROCESSAMENTO DE DISPOSITIVOS MÉDICOS REUTILIZÁVEIS
- SK OPÄŤOVNÉ OPRACOVANIE VIACRÁZOVO POUŽITELNÝCH LEKÁRSKYCH NÁSTROJOV
- SL OBDELAVA PONOVO UPORABLJIVIH MEDICINSKIH PRIPOMOČKOV
- SV BEARBETNING AV ÅTERANVÄNDBARA MEDICINSKA APPARAT
- TR YENİDEN KULLANILABİLİR TIBBİ CİHAZLARIN YENİDEN İŞLENMESİ

Sales & Customer Service

Tel: 01451 810900 · Fax: 01451 822302

e-mail: info@summit-medical.co.uk

www.summit-medical.co.uk

S U M M I T M E D I C A L

STD-04

H724

Figure 1 | Figure 1 | Abbildung 1 | Figura 1 | Figura 1 | Obrázek 1 | Figur 1 | Afbeelding 1 | Joonis 1 | Kuva 1
Σχήμα 1 | 1. ábra | 1. attēls | 1 paveikšlītis | Figura Nru. 1 | Figur 1 | Rysunek 1 | Figura 1 | Obrázok 1 | Silka 1 | Figur 1 | Şekil 1

Unscrew locking collar | Deserrer le collier de blocage | Lösen Sie den Sicherungsring | Svitare il collare di bloccaggio
Desenosc ar el collarin de bloqneo | Odšroubujte stavěcí kroužek | Skru lāsakraven af | Schroef de Halsring los
Keerata lahti lukustusmehhanisim. | Avaa lukitusholkki | Ëgbiðortu to koldpo aðqdaðunq. | Csavarja ki a zárgallért
Atskruvæt sledzenes siksaniñu | Atsukite riksuojanši sandariki | Holl it-kamin ta' l-qhong li jaghlaqha. | Skru los lāseringen
Odkreć wkręt pierścienia blokującego | Desaparafusar a luva de bloqneo | Odskrutkujcie uzamykaciu objimku
Odvijte zapirahni obroček | Skruva av laskragen | Kilitleme bileziğini çıkartın

REPROCESSING OF REUSABLE MEDICAL DEVICES

INSTRUCTIONS FOR CLEANING & STERILISING

SUMMIT MEDICAL RE-USABLE SURGICAL INSTRUMENTS

CLEANING

H716, H719, H749F, H759 & H759L
Disassembly not required.

H724

Unscrew locking collar.
Please refer to figure 1 for product diagrams.

PRE-WASH

Carry out either A) ultrasonic or B) manual, as below.

A) ULTRASONIC

Place device in an ultrasonic tank containing an enzymatic detergent for 3 minutes.

B) MANUAL

Fill a sink with water from a tap. Submerge the device and brush using an enzymatic detergent.

CLEANING - DISINFECTING

Carry out either A) automatic or B) manual, as below.

A) AUTOMATIC

- 1) Load instruments in to an automated washer-disinfector and run according to the instructions.

- 2) Use with a general-purpose pH neutral detergent.

3) For disinfection, thermal contact times should be in accordance with national guidelines. (In the UK, refer to HTM2030 i.e. 80°C for 1 minute or 90°C for 12 seconds).

- 4) Ensure all visible contamination is removed. If necessary, repeat cycle or use manual cleaning.

- 5) Ensure the device is thoroughly dry following the cleaning-disinfecting stage.

B) MANUAL

- 1) Fill a sink with water from a tap. Submerge the device and brush using a general-purpose pH neutral detergent.

threads and blind holes. Refer to national guidelines on manual cleaning. (In the UK, refer to MDA publication on manual cleaning (Guidance on decontamination)).

- 2) Ensure all visible contamination is removed. If necessary, repeat part 1.

- 3) For chemical disinfection, use a broad-spectrum, pH neutral disinfectant according to the label instructions.

- 4) Ensure the device is thoroughly dry following the cleaning-disinfection stage.

INSPECTION

- 1) Visually inspect the device for damage and wear (e.g. snags and burrs). Pay particular attention to ratchet teeth and threads.

- 2) With the cement delivery guns (H716, H719, H749F, H759 & H759L), ensure that the shaft advances forward with each squeeze of the trigger.

PACKAGING

- 1) Place the device in to a sterilisation tray.

- 2) Wrap the trays in a medical grade sterilisation wrap.

3) Refer to EN868 'Packaging materials and systems'.

STERILISATION

- 1) The device should be sterilised in a pre-vacuum/porous load steam steriliser at 134°C-137°C for a minimum of 3 to 3½ minutes or at 121°C for a minimum of 15 minutes.

- 2) Do not sterilise the device using any method other than steam.

NUMBER OF REUSES

Wear and damage during use determine the number of reuses of this device. Repeated processing has a minimal effect on these instruments.

SPECIAL PRECAUTIONS

- 1) Do not use alkaline detergents and solutions (>pH10) on aluminium based instruments such as cement delivery guns H716, H719, H749F, H759 & H759L.

H759 & H759L.

- 2) If the instruments come in to contact with a patient at risk from Transmissible Spongiform Encephalopathy, precautions should be taken in line with national guidelines. (In the UK, refer to ACDP/SEAC document 'TSE agents: safe working and the prevention of infection').

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